

INSTRUCTIONS FOR USING HYBRID AL40 NOMAD MEDICAL COOL BOX.

To achieve the best results from your NOMAD Cool Box, please follow the instructions below very carefully:

1. Use a domestic freezer at temperature of -15°C or lower is recommended, always clean the gel packs first, freeze the gel packs, approximately **12 hours**, keep them as flat as possible and away from the side walls and floor of the freezer, on a plastic tray for best long term results.
2. Once frozen, remove from the freezer, and place them in the pockets of the cool box leave the lid open in a room temperature of circa 22°C .



3. Place the cleaned thermometer probe into one of the pockets, then observe the thermometer until it reads $+2^{\circ}\text{C}$.
4. As soon as the temperature reaches $+2^{\circ}\text{C}$ load your medication/vaccines into the cooler and close the lid.
5. The cooler will hold between 2° and 8°C for the number of hours stated in description of model.
6. When not in use, thoroughly clean down the box and gel packs with hot soapy water. Avoid water getting inside the Thermometer.

Note:

The coolant is non-toxic and non-hazardous – DO NOT EAT, if ingested, drink plenty of water and seek medical advise. For full technical information this can be downloaded from our website.

If you require any further help, please use our email address; info@coolicebox.com

If its an emergency please use the following number; 0843 277 770