


NOMAD TRAVEL CASE EXTRA LARGE – Product code - NTCEX Insulin Cooler Bag User Instructions

Insulin cooler bags are designed for easy carriage and requires no power

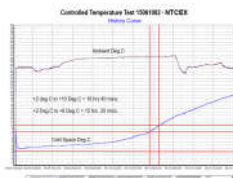

Instructions for use

The Insulin Cooler Bags when used with the appropriate gel packs, can cool items down to approximately 2 °C to 8 °C for approximately 6-12 hours depending on the model.

Carefully place the ice gel packs into a freezer at a temperature of approximately -11 °C to -15 °C for a minimum of 24 hours. Ensure there is no direct contact between the ice gel packs and the freezer walls.



How to care for your NTCEX Insulin Cooler Bag

1. Outdoors: Keep the Insulin Cooler Bag out of direct sunlight.
2. Indoors: Keep the Insulin Cooler Bag away from any source of high temperature.
3. In the car: Do not place the Insulin Cooler Bag next to heaters or close to the engine compartment.
4. Never leave the Insulin Cooler Bag zipper open. Keep open for as short a time as possible.
5. The Insulin Cooler Bag is safe to be carried in other bags or boxes.
6. Do not try to open the ice gel packs. Avoid damage to the ice gel packs from sharp edges or sharp implements.
7. Do not place the ice gel packs in direct contact with the freezer walls
8. Do not attempt to fold the gel ice packs
9. The gel contained in the ice packs is non-toxic and eco-friendly. Always discard unusable gel from damaged ice gel packs in accordance with the regulations for protecting the environment in your State or Country.
10. Do not eat the gel. If ingested by accident, rinse the mouth out thoroughly with water, then drink as much water as possible. Be sure to see a doctor immediately for examination.
11. Do not use ice gel packs that have been damaged.
12. To prolong the life of the zip careful use is required. If the zip becomes stiff a small amount of Vaseline or DW40 can be applied.
12. It is highly recommended that the product is tried and tested before use.