

NOMAD TRAVEL CASE INSTRUCTIONS FOR USE

Application

Travel cases are very lightweight, and easy to operate since they do not use electricity. If your medications are placed in the case at room temperature, our cooler can keep the temperature between 2-8C for 6-12 hours depending on the model chosen.

How to use it?

Before using the case, wipe the gel packs dry, then place them in a freezer (-5~-10C). Make sure the gel packs are flat without wrinkles. They should be frozen after 12 hours.

A tip - When you are not using your travel case, store your gel packs in the freezer so that they are ready for the next time you want to use them.

Remove the frozen gel packs from your freezer and place a gel brick in each pocket of the liner.

Secure the liner into the case using the Velcro strips that are attached to the liner inside the case, if it has any.

Place your medications/vaccines inside the case and close the zipper.

Caution!

Do not attempt to bend the frozen gel packs - this may cause the outer plastic bag surrounding the gel packs to rupture.

Be careful that sharp objects do not penetrate your gel packs.

Gel packs are not toxic. But in case a pack is accidentally eaten, you should rinse your mouth with water and then drink as much water as possible. Remember to consult your doctor immediately for an examination.

