

INSTRUCTIONS FOR USE; NOMAD MEDICAL

FROZEN SERUM EYE COOLER NTC1714

To achieve the best results from your NOMAD NTC1714, please follow the instructions below very carefully:

1. Freeze the gel packs for 24 hours, they will turn clear to a milky colour. Lay them flat on a tray or shelf in your freezer. Ensure that they do not touch the side or any metal racking in the freezer. A household freezer runs at -19oc.
2. Once fully frozen after 24 hours, take out of the freezer and Place 1 pack inside on the bottom on the NTC1714, 1 coolant pack each side, then place the medication in the middle, if need be wrap the medication in bubble wrap to fill all the air space and then place the last pack on the top, and the rectangular pad on top.
3. Close the lid down and off you go.



Our test have stayed frozen for between 8 to 12 hours at 20oc ambient temperature.

If you require any further help, please use our email address; info@coolicebox.com

If its an emergency please use the following number;
01769 610 170